



MULTIPURPOSE OVERBED TABLE

User's Guide



AFTER SALES SUPPORT



USA

(800) 407 1458

ITEM #44576

Contents

01 Introduction	
Intended use	p. 04
02 General Information and Safety Instructions	
Technical Data	p. 04 - 05
Materials	p. 05
Safety Instructions	p. 05 - 06
03 Assembly and Parts List	
Parts List	p. 06
Assembly	p. 07 - 08
04 Other Useful Information	
Cleaning and storage	p. 09
Disposal	p. 09
After Sales Service	p. 09



NOTE
Pay attention to important information and notes when reading the manual.



ATTENTION
Contains important information to avoid the risk of damage to the product.



WARNING
Contains important information to avoid personal injury.

Introduction

1

Congratulations on choosing to buy a WELBY HEALTH product. All products brought to you by WELBY HEALTH are manufactured to the highest standards of performance and safety and, as part of our philosophy of customer service and satisfaction, are backed by our comprehensive 1 year warranty.

Please fill in and return the Warranty Card to the address provided.

We hope you will enjoy using your purchase for many years to come.



Important note

Please read the User's Guide carefully before using your product. It is imperative that you keep the User's Guide for future reference!

Intended use

The item is designed for private use as a side table for a maximum load up to 10 kg (22 lbs). The item is intended for use in the home and not for use outdoors.

General Information and Safety Instructions

2

Technical Data:

Dimensions when assembled: approx. 30 x 24 x 36.4 inches (L/W/H)
approx. 76.2 x 61 x 92.4 cm (L/W/H)

Table top: approx. 23.6 x 15.7 inches
approx. 60 x 40 cm

Weight limit of table top: 22 lbs
10 kg

Table leg height adjustable: from approx. 28 to 36.2 inches
from approx. 71 to 92 cm

Product weight without packaging: approx. 19 lbs
approx. 8.6 kg

Wheels: diameter approx. 1.6 inches
diameter approx. 4 cm

Materials:

Base: steel
Table top: particle board
Wheels: nylon

Safety Instructions:



Please observe the following to avoid any personal injury and /or damage to the product:

- Please read the instructions for use carefully before using the table.
- Do not exceed the maximum weight limit of table top: 22 lbs / 10 kg.
- Do not sit or stand on the table.
- Do not use the table for a purpose for which it is not intended.
- The table is designed for private use only.
- Any misuse may result in personal injury and / or damage to the product.
- Keep children away from the table. The table may tip over when children sit on it or use it to pull themselves upright.
- Only an adult may assemble the table. Pay attention and do not crush any part of your body when assembling the table.
- Screw the individual parts in tightly before using the table.
- Before using the table, always carry out a visual inspection beforehand in order to check for any faults, such as damages. If you find any damage, have the table repaired before use.
- Do not carry out any modifications to this table.
- This User's Guide applies to the use of the product throughout its lifetime. You must ensure that this User's Guide is kept together with the table in the event of it being passed on to somebody else.
- Once you have positioned the table, activate the brakes on the two wheels.



Pay attention and do not crush any part of your body when assembling the table.

Assembly and Parts List

3

Parts List:


- 4 x Wheels (2 x wheels with brakes)
- 2 x Screws with washers
- 1 x Wrench
- 1 x Table top
- 1 x Base frame
- 1 x Table leg
- 1 x User's Guide



Assembly:



Step 1

Put the wheels with brakes into the openings on the two small protruding parts on the front of the frame (1) and the other two wheels into the openings on the end parts of frame (2). Make sure the wheels are fixed securely in the openings.

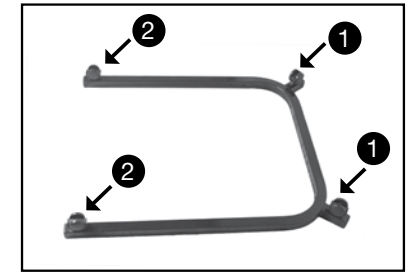


Step 2

Turn the base frame over and place the bottom part of table leg onto the base frame where the two holes are (1). Use the screws to fix the bottom part of table leg onto the base frame (2). Make sure that the washers are in place before fixing the nut with the wrench. Make sure the screws and nuts are fixed tightly.

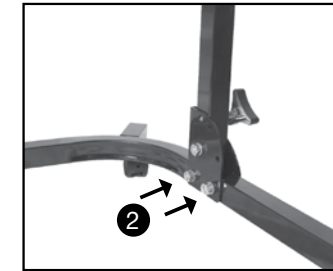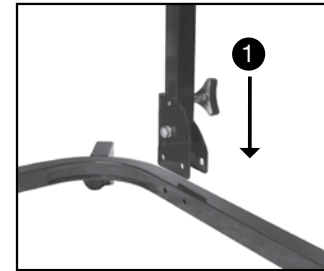


Step 3

Insert the silver colored end of the table leg into the pipe on the underside of the table top. Make sure the button on the silver colored tube is pressed down when doing so. This button pops out at the other end of the pipe when the silver colored tube is inserted correctly.

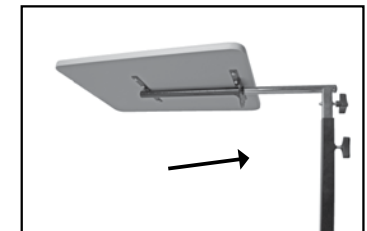

Step 4

To finally fix the table top position, the black adjustment knob (1) has to be screwed through one of the holes in the fixing device on the underside of the table top (2).

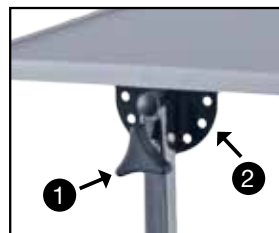

Adjusting the tilt angles and the table height



1 **Adjusting the tilt angle of the table top**

Loosen the upper adjustment knob (1) and tilt the table top until in the desired position. Fix the new position by screwing the adjustment knob through one of the holes in the fixing device (2).

2 **Adjusting the tilt angle of the table leg**

Loosen the lower adjustment knob (3) until the table leg can be tilted into another position. Then screw the knob through the matching hole in the fixing device on lower part of table leg until it is securely fixed.

3 **Adjusting the table height**

Loosen the knob on the black part of table leg (4) so that the silver colored part can be moved. Move the silver colored part up or down until the desired height is achieved. Then tighten the knob until the silver colored part is fixed securely.

Other Useful Information

Cleaning and storage



Do not use any corrosive or abrasive cleaners or sponges. These may damage the surface. When needed, wipe the item with a slightly damp cloth and some mild detergent if required. Then dry the item completely before using it again.



Check the screw connections regularly.

Disposal

Please dispose of waste in an environmentally safe manner.

After Sales Service:

US HELPLINE NUMBER: (800) 407 1458
EMAIL: SUPPORT@ASPIRIA-NONFOOD.COM



_____ Multipurpose Overbed Table _____

1 YEAR WARRANTY CARD

Name _____

Address _____

Email _____ Tel No _____

Item No. 44576

Location of purchase _____

Please send your completed Warranty Card to:

USA

Joachim Pfahler
1224 Coronado Ave.
Spring Valley, CA 91977

DISTRIBUTED BY ALDI, INC.
BATAVIA, IL 60510
WWW.ALDI.US | MADE IN CHINA